



CLEAR BLUE SMILES™

SureSmile Aligner

#6 Retention

Appointment #4

OVERVIEW

After treatment has been completed, retainers will be ordered.

PREPARATION

Review handout:
#6 Retention

APPOINTMENT

Appointment time commitment:
15 minutes

CLEAR BLUE SMILES™



Option #1- Retention Workflow

3

1. Login into SureSmile®

Dentsply Sirona SureSmile®

Log in with your account details

Email address

Password

Login

[Forgot your password?](#) [Looking to add a new user to your account?](#) [New to SureSmile*? Click here to create an account](#)

© 2022 OraMetrix. All rights reserved. [Privacy Policy](#)

CLEAR BLUE SMILES™

***Login into SureSmile and land on your dashboard**

2. Select Review setup and staged models

AuraSmile Dental

Dr. Brian Gaudreault
AuraSmile Dental

SureSmile Aligner

Sort by

Task

Show Tasks of

All Types

Tasks for

All

View

Tasks (4)

Patients

Shipments

Jobs

DS CEREC Orders


Reports

Active Tasks

Flag	Card	Last name	First name	Patient ID	Owner	Item	Task	Due Date	Notes	Setup Approval Date	Status	Deferred To
<input type="checkbox"/>		Walker	Shekeda	O3P3	Douglas W. Voiers, DDS		Order Retainers	2022-07-04		2022-06-16	Active	
<input checked="" type="checkbox"/>		Dillon	Sanders	H5Y4	Douglas W. Voiers, DDS		Review setup & staged models	2022-06-30		2022-07-27	Active	
<input type="checkbox"/>		Wayne	Renee	V9T8	Douglas W. Voiers, DDS		Review setup & staged models	2022-06-30		2022-07-20	Active	
<input type="checkbox"/>		Miller	Jeff	G6M8	Douglas W. Voiers, DDS		Review setup & staged models	2022-07-01		2022-07-21	Active	

CLEAR BLUE SMILES™

3. Select Order Retainers

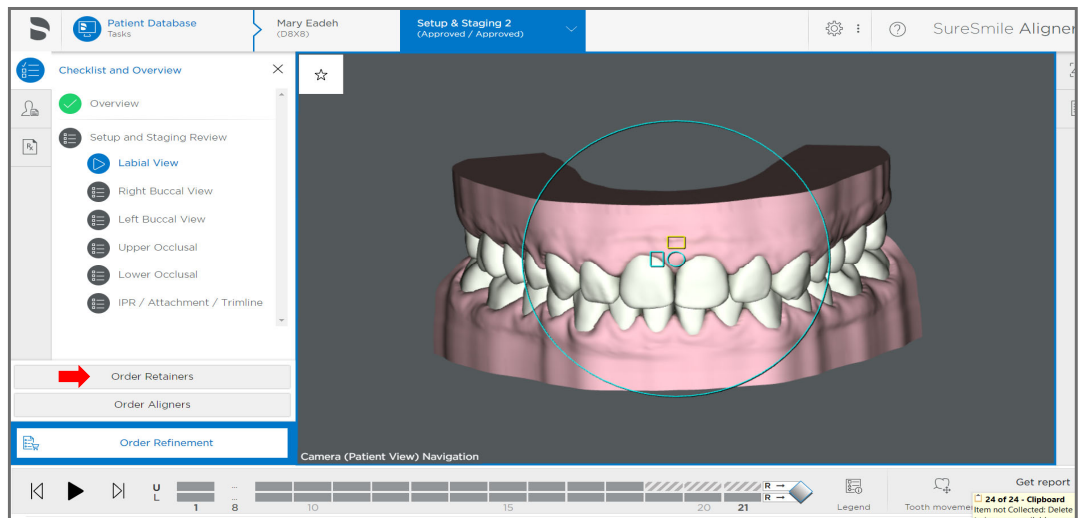

CLEAR BLUE SMILES™

4. Select Arches



Order Retainers ⓘ

☒ Upper

☒ Lower

Stage R ▾

1 ▴ ▾

Close

Next

CLEAR BLUE SMILES™

Select Arches



Order Retainers ⓘ

☒ Upper

☒ Lower

Stage R

▼

1

▲▼

Close

Next

CLEAR BLUE SMILES™

5. Select Number of Retainers

Order Retainers ⓘ

Upper

Lower

Stage R

1

Close

Next

CLEAR BLUE SMILES™

6. Select Next

Order Retainers ⓘ

Upper

Lower

Stage R

1

Close

Next



CLEAR BLUE SMILES™

7. Select Place Order

Order Retainers

×


This order will incur a one-time charge of 60.00 USD.

Place Order



CLEAR BLUE SMILES™

8. Select OK

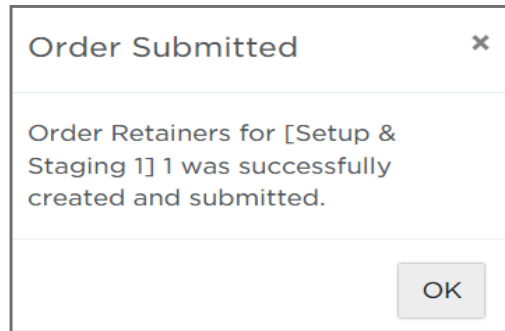


CLEAR BLUE SMILES™

A decorative graphic at the top of the slide featuring a blue background with white, stylized, wavy shapes that resemble water or clouds.

Option #2- Retention Workflow

1. Login into SureSmile®

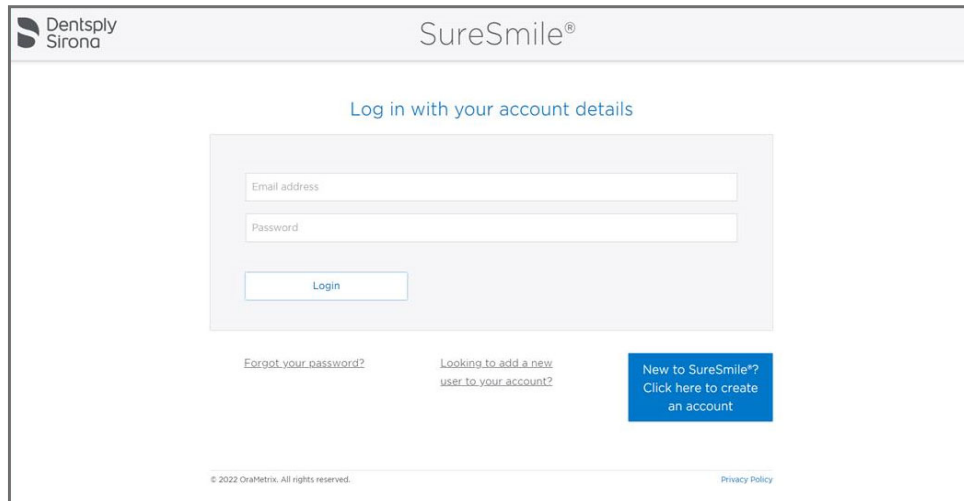

The screenshot shows the SureSmile login interface. At the top left is the Dentsply Sirona logo, and at the top right is the SureSmile® logo. Below the logos, the text "Log in with your account details" is centered. A central form contains two input fields: "Email address" and "Password", followed by a "Login" button. Below the form, there are two links: "Forgot your password?" and "Looking to add a new user to your account?". To the right of these links is a blue button that says "New to SureSmile? Click here to create an account". At the bottom of the page, there is a copyright notice "© 2022 OraMetrix. All rights reserved." and a "Privacy Policy" link.

CLEAR BLUE SMILES™

***Login into SureSmile and land on your dashboard**

2. Select Order Retainers

AuraSmile Dental

Dr. Brian Gaudreault
AuraSmile Dental

SureSmile Aligner

Sort by

Task

Show Tasks of

All Types

Tasks for

All

View

Table View

Grid View

Tasks (4)

Patients

Shipments

Jobs

DS CEREC Orders


Reports

Active Tasks

Flag	Card	Last name	First name	Patient ID	Owner	Item	Task	Due Date	Notes	Setup Approval Date	Status	Deferred To
<input type="checkbox"/>		Walker	Shekeda	O3P3	Douglas W. Voiers, DDS		Order Retainers	2022-07-04		2022-06-16	Active	
<input checked="" type="checkbox"/>		Dillon	Sanders	H5Y4	Douglas W. Voiers, DDS		Review setup & staged models	2022-06-30		2022-07-27	Active	
<input type="checkbox"/>		Wayne	Renee	V9T8	Douglas W. Voiers, DDS		Review setup & staged models	2022-06-30		2022-07-20	Active	
<input type="checkbox"/>		Miller	Jeff	G6M8	Douglas W. Voiers, DDS		Review setup & staged models	2022-07-01		2022-07-21	Active	

CLEAR BLUE SMILES™

3. Select Order Retainers

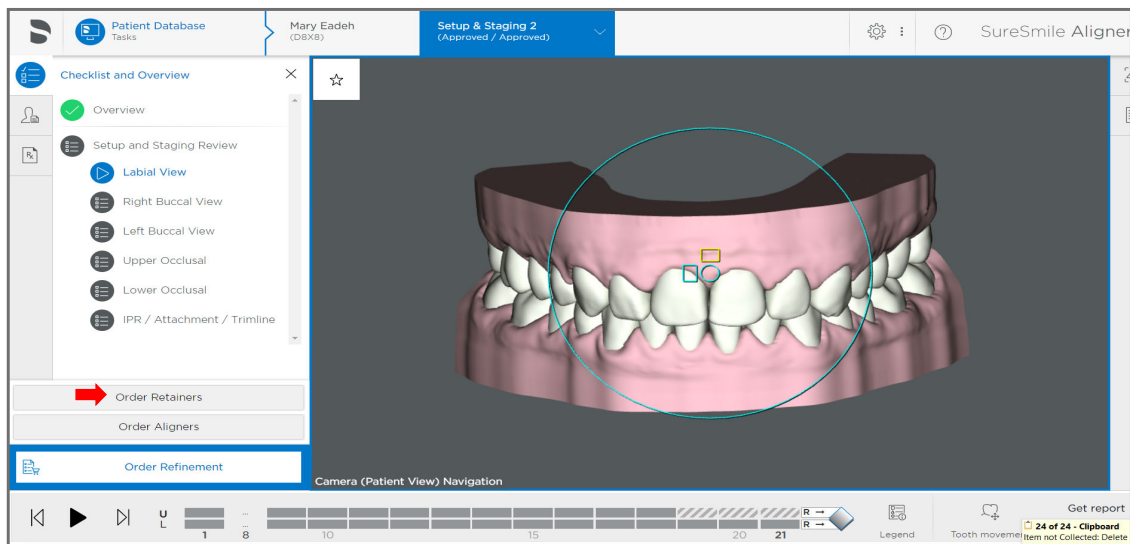

CLEAR BLUE SMILES™

4. Select Arches



Order Retainers ⓘ

☒ Upper

☒ Lower

Stage R

▼

1

▲▼

Close

Next

CLEAR BLUE SMILES™

Select Arches



Order Retainers ⓘ

☒ Upper

☒ Lower

Stage R

▼

1

▲▼

Close

Next

CLEAR BLUE SMILES™

5. Select Number of Retainers

Order Retainers ⓘ

Upper

Lower

Stage R

1

6. Select Next

Order Retainers ⓘ

☒ Upper

☒ Lower

Stage R ▾

1 ▴ ▾

Close

Next



CLEAR BLUE SMILES™

7. Select Place Order

Order Retainers

×


This order will incur a one-time charge of 60.00 USD.

Place Order



CLEAR BLUE SMILES™

8. Select OK

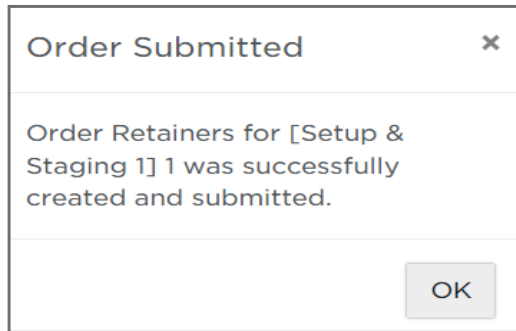

CLEAR BLUE SMILES™