




CLEAR BLUE SMILES™

Suresmile® Aligner

My iTero-Export File

1. Login into My iTero

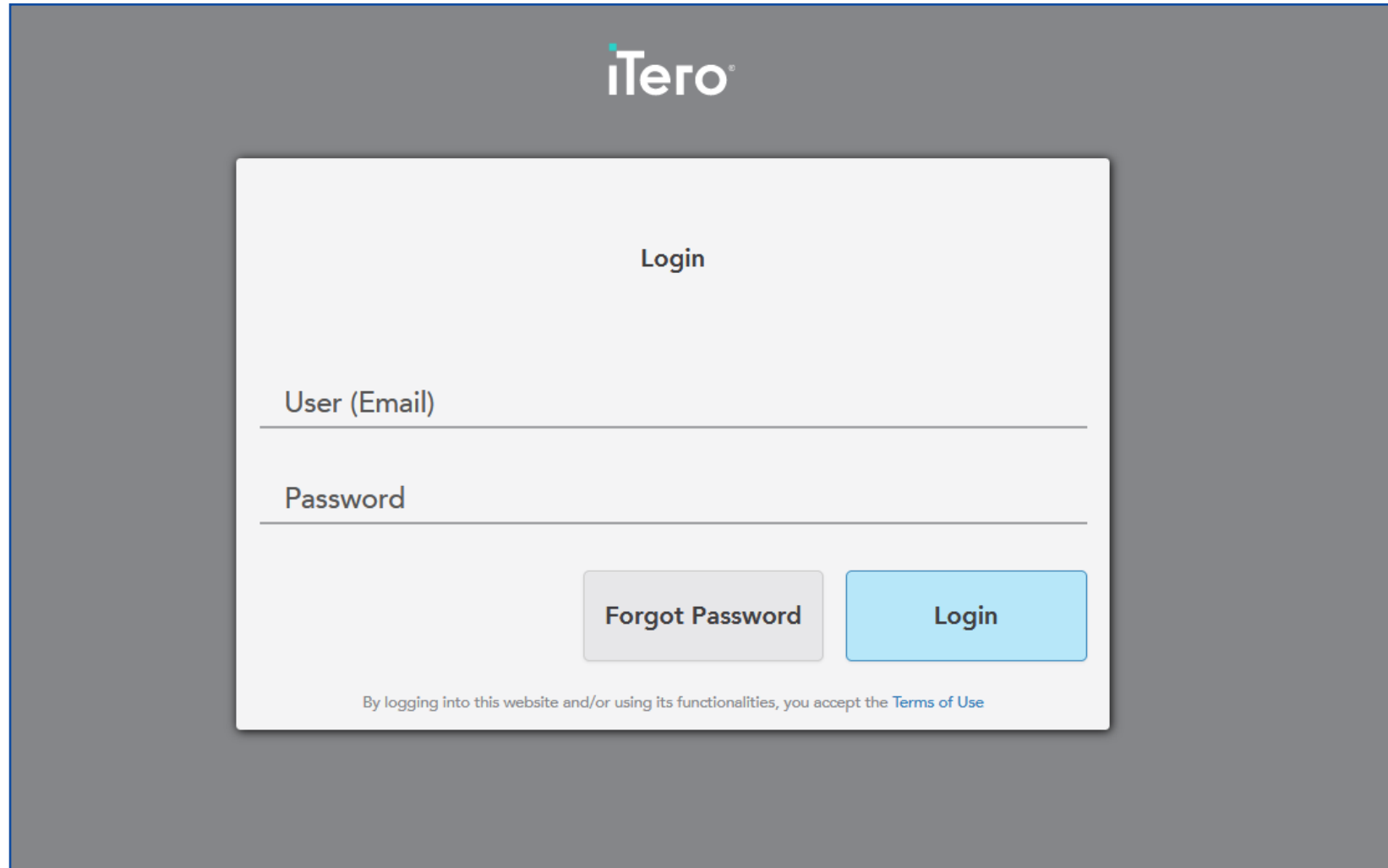

The screenshot shows the iTero login interface. At the top center is the iTero logo. Below it is the word "Login". There are two input fields: "User (Email)" and "Password". Below the input fields are two buttons: "Forgot Password" and "Login". At the bottom of the form is a line of text: "By logging into this website and/or using its functionalities, you accept the Terms of Use".

iTero[®]

Login


User (Email)

Password

Forgot Password Login

By logging into this website and/or using its functionalities, you accept the Terms of Use

2. Select Patients

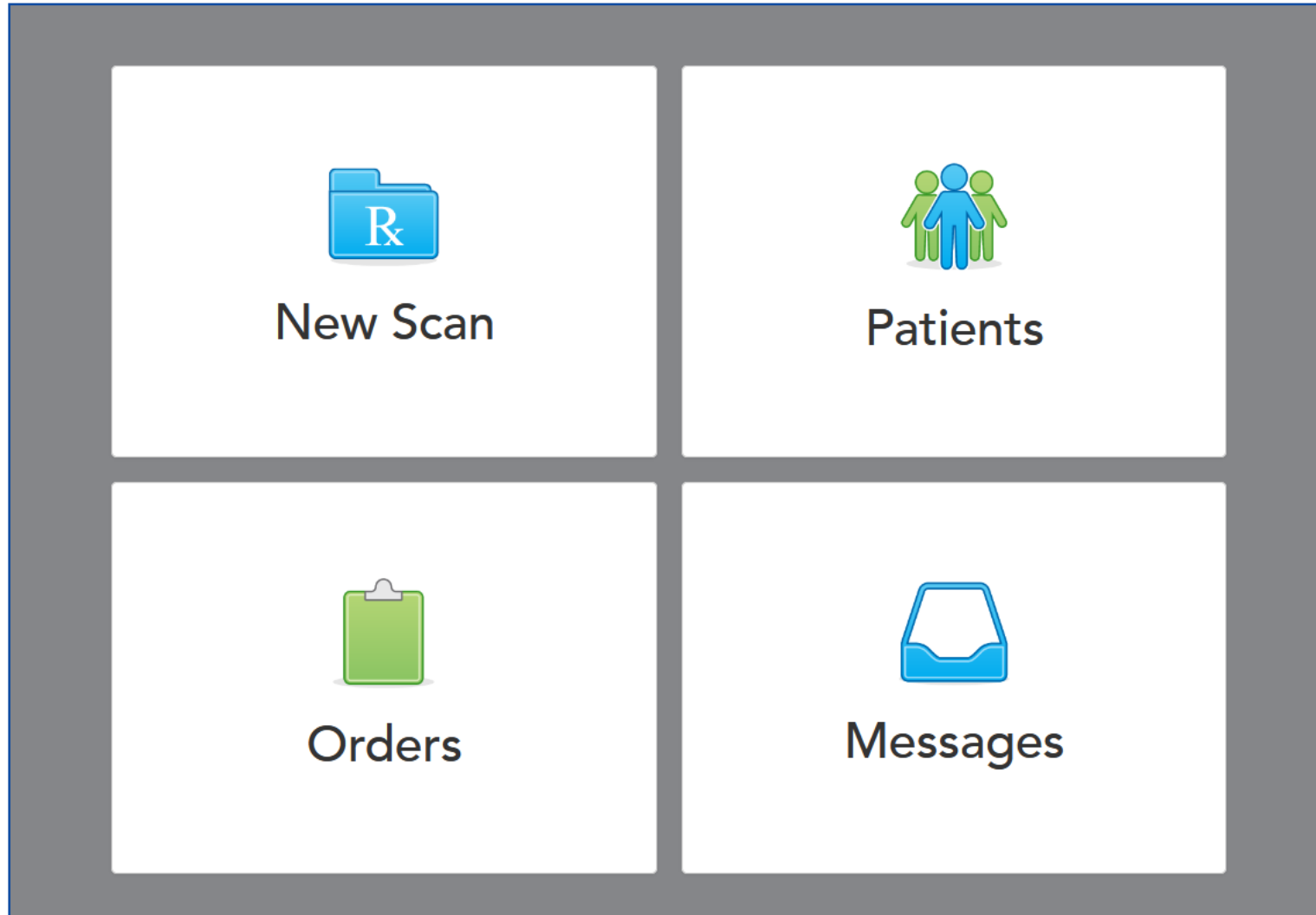

3. Search Patient



Name ▲	Chart Number	Date Of Birth	Last Scan Date
Abbott, David	2226408M		02/24/2022
Ackerman, Ryan	178475m		08/02/2022
Ackland, Jolynn	178476M		12/19/2019
Adams, Ava	178481m		09/19/2019

4. Select Patient

 Patients

 Southeast Orthodontics (Raynham)

 Dr. Gaudreault, Brian



Name ▲

Chart Number

Date Of Birth

Last Scan Date

Holland, Courtney

1820326M

05/09/2019

Holland, David

1719164M

02/18/2020

Holmes, Toni 

10/15/2020

5. Select Order

< Patient: Holmes, Toni



Southeast Orthodontics (Raynham)



Dr. Gaudreault, Brian



Patient Name
Holmes, Toni



New Scan

Orders

ID	Scan Date	Case Type	Status
42903348	10/15/2020	Invisalign + iRecord	Completed



6. Select Export

← Patient: Holmes, Toni

Southeast Orthodontics (Raynham) Dr. Gaudreault, Brian

Patient Name
Holmes, Toni


New Scan

Orders

ID	Scan Date	Case Type	Status
42903348	10/15/2020	Invisalign + iRecord	Completed

View Rx Change Case Type Viewer **Export** iTero Scan Report Invisalign Outcome Simulator

Invisalign Progress Assessment OrthoCAD



7. Select Export Type- Open Shell

Export Type	Open Shell	▼
Data Format	File per Arch (arches oriented in occl...	▼
File Type	STL (No color)	▼
<input type="checkbox"/> Hide patient name		
	Cancel	Export



8. Select Data Format- File per Arch (arches oriented in occlusion)

Export Type	Open Shell	▼
Data Format	File per Arch (arches oriented in occl...	▼
File Type	STL (No color)	▼
<input type="checkbox"/> Hide patient name		
	Cancel	Export



9. Select File Type- STL (No color)

Export Type	Open Shell	▼
Data Format	File per Arch (arches oriented in occl...	▼
File Type	STL (No color)	▼
<input type="checkbox"/> Hide patient name		
	Cancel	Export




10. Select Export

Export Type	Open Shell	▼
Data Format	File per Arch (arches oriented in occl...	▼
File Type	STL (No color)	▼
<input type="checkbox"/> Hide patient name		
	<input type="button" value="Cancel"/>	<input type="button" value="Export"/>




11. Save File

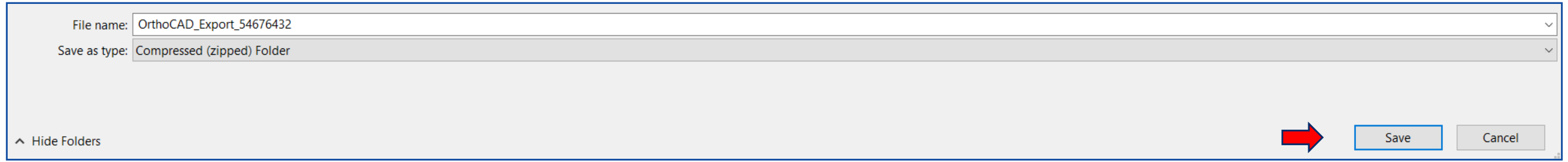

- Choose destination Folder-Save

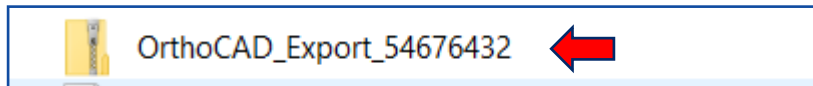


- Select Folder



- Select Exported Files-Ready for Upload



Name	Type	Compressed size	Password p...	Size	Ratio	Date modified
 54676432_shell_occlusion_l	STL File	13,859 KB	No	13,859 KB	0%	11/23/2022 6:32 PM
 54676432_shell_occlusion_u	STL File	16,207 KB	No	16,207 KB	0%	11/23/2022 6:32 PM

6. Select Export

Name	Type	Compressed size	Password p...	Size	Ratio	Date modified
 54676432_shell_occlusion_l	STL File	13,859 KB	No	13,859 KB	0%	11/23/2022 6:32 PM
 54676432_shell_occlusion_u	STL File	16,207 KB	No	16,207 KB	0%	11/23/2022 6:32 PM

Quick access

- Desktop
- Downloads

Name	Date modified	Type	Size
 OrthoCAD_Export_54676432			
 13081_E4D6_Staged_Models_1	11/9/2022 9:22 PM	Compressed (zipp...	3,533 KB


File name: OrthoCAD_Export_54676432



Save as type: Compressed (zipped) Folder

Save

Cancel

6. Select Export

Name	Type	Compressed size	Password p...	Size	Ratio	Date modified
 54676432_shell_occlusion_l	STL File	13,859 KB	No	13,859 KB	0%	11/23/2022 6:32 PM
 54676432_shell_occlusion_u	STL File	16,207 KB	No	16,207 KB	0%	11/23/2022 6:32 PM

Quick access	Name	Date modified	Type	Size
Desktop	 OrthoCAD_Export_54676432			
Downloads	 13081_E4D6_Staged_Models_1	11/9/2022 9:22 PM	Compressed (zipp...	3,533 KB

File name: OrthoCAD_Export_54676432

Save as type: Compressed (zipped) Folder

Hide Folders

Save Cancel