



CLEAR BLUE SMILES™

Suresmile® Aligner

Replacement Aligners

1. Login into SureSmile

Dentsply Sirona

SureSmile®

Log in with your account details

Email address

Password

Login

[Forgot your password?](#)

[Looking to add a new user to your account?](#)


New to SureSmile®?
Click here to create an account

© 2022 OraMetrix. All rights reserved.

[Privacy Policy](#)



2. Select on Patients tab

Patient Database Tasks





Patient ID  [New Patient](#)

Dr. Brian Gaudreault
SOUTHEAST ORTHO-ALIGNER

SureSmile Aligner

Sort by: Task Show Tasks of: All Types Tasks for: Dr. Brian Gau... View:  

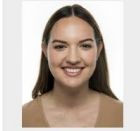

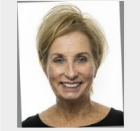






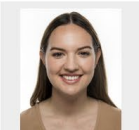


Tasks (1) Patients Shipments Jobs 3Shape Cases Reports Active Tasks

Flag	Card	Last name	First name	Patient ID	Owner	Item	Task	Due Date	Notes	Setup Approval Date	Status	Deferred To
		Hudzik	Anna	G8S7	Dr. Brian Gaudreault		Review setup & staged models 	2022-08-01		2022-08-19	Active	

3. Select on Setup & Staging

Tasks Patients (57) Shipments Jobs Reports ?

Sort by: Modification ... Patients for: All Show: Active Patien... View: [List View] [Grid View]

 ID: R3F3	 ID: E8J5	 ID: W4X4 Setup & Staging 1 2022-09-29 Approved / Approved	 ID: N4G6
 ID: S6X2 Setup & Staging 1 2021-08-30 Approved / Approved	 ID: T2X7	 ID: K4H6	 ID: S7H8
 ID: R3E7	 ID: L6K9	 ID: K8J7	 ID: Y9F9

4. Select Order Aligners

The screenshot displays the SureSmile Aligner software interface. At the top, the patient information is shown as "Anna Hudzik (G8S7)" and the treatment stage is "Setup & Staging 1 (Approved / Approved)". The main window shows a 3D model of a patient's teeth in a patient view, with a red circle highlighting the upper front teeth. A red arrow points to the "Order Aligners" button in the left sidebar. The sidebar also includes "Order Retainers" and "Order Refinement" buttons. The bottom of the interface features a navigation bar with playback controls, a timeline from 1 to 29, and a "Get report" button.

Patient Database Tasks

Anna Hudzik (G8S7)

Setup & Staging 1 (Approved / Approved)

SureSmile Aligner

Checklist and Overview

Overview

Click the Notes Tab and review the notes from the Digital Lab (DL). Go to next step to begin reviewing the setup in different views, and review IPR

Setup and Staging Review

- Labial View
- Right Buccal View
- Left Buccal View
- Upper Occlusal
- Lower Occlusal

Order Retainers

Order Aligners

Order Refinement

Camera (Patient View) Navigation

1 16 20 25 29

Legend

Tooth movement

Get report

Order Aligners Opens

Order Aligners ✕

Upper

1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
17	18	19	20	21	22	23	24	25	26	27	28				

Lower

1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
17	18	19	20	21	22	23	24	25	26	27	28				

Please select at least one stage.

Legend

SureSmile® Retainers can be ordered any time after placing your initial aligner order. A reminder to "Order Retainers" will appear in your task list 6 weeks prior to end of treatment if retainers have not previously been ordered.

Trim Type & Height Close Next

Select Legend

Order Aligners ×

Upper

1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
17	18	19	20	21	22	23	24	25	26	27	28				

Lower

1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
17	18	19	20	21	22	23	24	25	26	27	28				

Please select at least one stage.

Legend

not selected	selected	no movement	conflict	printed
--------------	----------	-------------	----------	---------

SureSmile® Retainers can be ordered any time after placing your initial aligner order. A reminder to "Order Retainers" will appear in your task list 6 weeks prior to end of treatment if retainers have not previously been ordered.

Trim Type & Height Close Next



5. Select Aligners

Order Aligners ✕

Upper

1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
17	18	19	20	21	22	23	24	25	26	27	28				

Lower

1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
17	18	19	20	21	22	23	24	25	26	27	28				

Legend

not selected	selected	no movement	conflict	printed
--------------	----------	-------------	----------	---------

SureSmile® Retainers can be ordered any time after placing your initial aligner order. A reminder to "Order Retainers" will appear in your task list 6 weeks prior to end of treatment if retainers have not previously been ordered.

Trim Type & Height Close Next

6. Select Next

Order Aligners ✕

Upper

1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
17	18	19	20	21	22	23	24	25	26	27	28				

Lower

1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
17	18	19	20	21	22	23	24	25	26	27	28				

Legend

not selected	selected	no movement	conflict	printed
--------------	----------	-------------	----------	---------

SureSmile® Retainers can be ordered any time after placing your initial aligner order. A reminder to "Order Retainers" will appear in your task list 6 weeks prior to end of treatment if retainers have not previously been ordered.

Trim Type & Height Close Next



7. Select OK

